Doküman No: IFU-GNC <i>Documentation No: IFU-GNC</i>	İlk yayın tarihi: 02/05/2012 <i>Valid from: 02/05/2012</i>	Revizyon no: 01 <i>Revision no: 01</i>	Sayfa:1 <i>Page:1</i>
		Revizyon tarihi: 26.05.2016 <i>Revision date:26.05.2016</i>	

GEOTEK® MALEKOT NEFRS. KATETERİ**Kullanma Kılavuzu****TR**

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GEOTEK® MALECOT NEPH.CATHETER**Instruction for Use****ENG**

It is a Malekot type drainage catheter with tail made by soft PU.

Indications: is intended for drainage of fluids from kidney due to kidney stone formation in the Kidney Ureter. Intended for one time use.

Contraindications: Serious infections, hematuria, incontinence

Side Effects: Kidney, upper bladder irritation caused by foreignbody, pain, infection, increased miction-frequency, hematuria

Cautions: This Kit should be used by only by physicians experienced in the use of such device.

Regular monitoring of location and free passage of catheter is necessary to evaluate nephrostomy efficiency.

Ethylene oxide sterilized. Non-pyrogenic. Check the package. If package damaged or opened do not use the products. Check the expiry date and the gauge. Do not use over expired products. Store in dry place, between 5 – 30 °C temperature, protect from sun light. Possible allergic reactions should be considered. For single Patient Use Only. Do not attempt to clean or resterilize this product. Disposal as a biohazard after use.

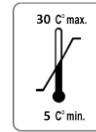
Placement of Nephrostomy Catheter:**Standard Seldinger technique:**

- Insert the Access needle under x-ray guidance in to renal pelvis
- Load the guide wire in to Access needle and push it to the renal pelvis
- Insert dilators in order, over the guide wire and complete dilatation
- Insert the Malecot catheter over guide wire and advance it to renal pelvis area. Please keep guide wire tight during this process.
- When the catheter placement is complete, withdraw the guide wire gently while holding the catheter at right position.
- Fix the catheter to skin with fixation disk.
- Connect the catheter with urine bag with bag connector.

Note: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

Single Use

Storage limits

Expiry Date

STERILE EO

Ethylene oxide sterilized

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GEOTEK MEDİKAL VE SAĞLIK HİZ.LTD.ŞTİ