Doküman No: IFU-GPNS Documentation No: IFU-GPNS	İlk yayın tarihi: 02/05/2012 Valid from: 02/05/2012	Revizyon no: 01 Revision no: 01	Sayfa:1 Page:1
		Revizyon tarihi: 26.05.2016 Revision date:26.05.2016	

GEOTEK® NEFROSTOMİ SETİ

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TR

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Instruction for Use

ENG

Nephrostomy set includes;Pigtail/Malekot type drainage catheter and 4 different sizes of dilators, PTFE coated J tip stiff guide wire, 18G Access Needle, Catheter fixation disk, bag connector and scalpel. Nephrostomy catheter made by soft PU.

Indications: Temporary External Drainage of renal collection sistem obstructed by inflammation, renal tumors..etc

Contraindications: Serious infections, hematuria, incontinence

Side Effects: Kidney, upper bladder irritation caused by foreignbody, pain, infection, increased miction-frequency, hematuria

Cautions: This Kit should be used by only by physicians experienced in the use of such device.

Regular monitoring of location and free passage of catheter is necessary to evaluate nephrostomy efficiency.

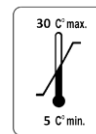
Ethylene oxide sterilized. Non-pyrogenic. Check the package. if package damaged or opened do not use the products. Check the expiry date and the gauge. Do not use over expired products. Store in dry place, between 5 – 30 °C temperature, protect from sun light.. Possible allergic reactions should be considered. For single Patient Use Only. Do not attempt to clean or resterilize this product. Disposal as a biohazard material after use.

Placement of Nephrostomy Catheter:

Standard Seldinger technique:

- Insert the Access needle under x-ray guidance in to renal pelvis
- Load the guide wire in to Access needle and push it to the renal pelvis
- Insert dilators in order, over the guide wire and complete dilatation
- Insert the Nephrostomy catheter over guide wire and advance it to renal pelvis area. Please keep guide wire tight during this process.
- When the catheter placement is complete, withdraw the guide wire gently while holding the catheter at right position.
- Fix the catheter to skin with fixation disk.
- Connect the catheter with urine bag with bag connector.

Note: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

STERILE EO

Single Use Storage limits Expiry Date Ethylene oxide sterilized

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GEOTEK MEDİKAL VE SAĞLIK HİZ.LTD.ŞTİ